

7 STEPS TO SAFETY WITH THE HOVERMATT®

BEFORE INFLATING HOVERMATT

1. Patient is informed regarding what to expect during transfer.



2. Patient is centered and secured on HoverMatt; At least two caregivers are present to move patient.



3. Transfer surfaces are as close as possible with **brakes locked on sending and receiving surface.**



FOR TRANSFER

4. Turn on Air Supply. Insert hose into hose entry (located at bottom of HoverMatt). Inflate mattress.



Single-Patient Use HoverMatt: Insert Air Supply hose at a 45-degree angle.

Reusable HoverMatt: Insert Air Supply hose straight-on.

5. Receiving staff member is alongside of receiving surface. Staff member on sending side pushes HoverMatt at an angle, either headfirst or feet first. Once half-way across, receiving caregiver should grasp closest handles and pull to desired location.



AFTER TRANSFER

6. Ensure patient is centered on receiving surface.



7. Turn off Air Supply to deflate HoverMatt and raise side rails.

