

7 STEPS TO SAFETY WITH THE HOVERJACK®

1. Place the deflated HoverJack® Air Patient Lift on the floor with Valve #1 against the floor.

*Depending on your model, ensure that the four (4) red deflate valve caps are screwed on and secure **OR** the four (4) Universal valves are closed.



2. To position the patient on the HoverJack, log-roll a HoverMatt® under the patient. Ensure patient is centered on HoverMatt. Insert air supply hose into either air intake at the foot-end of the HoverMatt. Press the appropriate button to inflate the HoverMatt.



3. Once inflated, move the patient/ HoverMatt onto deflated HoverJack. Make sure patient/ HoverMatt is centered on the HoverJack.



4. To inflate HoverJack, place air supply hose over the Valve #1, which is against the floor. Press the appropriate button on the air supply to initiate air flow. Ensure each chamber is fully inflated before inflating next chamber. Inflate chambers 2-4, as needed, in succession using the same technique.

LOOK: The patient's body will stop rising when chamber is fully inflated.
LISTEN: You will hear a pitch change.
FEEL: HoverJack will be hard to the touch.



5. Once fully inflated, secure HoverJack patient safety straps. Use transport handles to move inflated HoverJack to desired location. Before transferring patient using the HoverMatt, unbuckle HoverJack patient safety straps.



6. Insert air supply hose into either air intake at the foot-end of the HoverMatt. Press the appropriate button on the Air Supply to initiate airflow. Then transfer the patient by pushing the HoverMatt on an angle, either foot first or head first to the receiving surface.



7. Follow facility's cleaning protocol. For best results, leave HoverJack inflated for cleaning. Deflate HoverJack by removing the red caps or opening the Universal Valves. Once fully deflated, recap the 4 deflate valves or close the Universal Valves.

