

**Safe Patient Handling Equipment:**  
**HoverMatt® PROS™ Air**  
**SKILL CHECKLIST FOR ATTACHING LINK STRAPS TO BEDFRAME**

Equipment: HoverMatt PROS Air	<b>Name (print):</b>
Validator (print):	<b>Date:</b>
	<b>Patient Care Unit/Dept.:</b>

	<i>Performance Criteria</i>	<i>Validator Initials</i>
	Removes corner Velcro® connecting straps from pockets and loosely attaches to solid points on the bedframe or headboard to reduce matt migration.	
	Repeats the process on the other three corners of the matt.	
	Follows facility's protocol regarding raising the head of the bed and side rails.	
	Before safely boosting, turning, or transferring, disconnects connecting straps from bed frame. When straps are not in-use, stores in pockets.	
	If using one caregiver, raises the siderails prior to matt inflation.	
	Places the air supply at a convenient location.	
	Inserts the hose nozzle into either of two hose entries at foot end of PROS Air and snaps into place.	
	Presses the variable speed button to ensure the patient is centered.	
	When confirmed patient is centered, presses the 39" HoverMatt setting on the air supply to perform task.	

The above skills have been successfully demonstrated:	I agree with the Validator that I feel competent performing the above skills:
	<b>X</b>