

**Safe Patient Handling Equipment:
HoverMatt® PROS™ Air Sling
SKILL CHECKLIST FOR LOGROLLING**

Equipment: HoverMatt PROS Air Sling	Name (print):
Validator (print):	Date:
	Patient Care Unit/Dept.:

	<i>Performance Criteria</i>	<i>Validator Initials</i>
	Ensures the patient is supine and the bed is in a flat position.	
	Bends the patient's knee and rolls them onto their side.	
	Opens the product and places it lengthwise next to the patient.	
	Unfolds the PROS Air Sling furthest from the patient to the side of the bed.	
	Tucks the other side under the patient as far as possible.	
	Rolls the patient onto their other side.	
	Unrolls the rest of the sling from under the patient to cover the bed.	
	Places the patient back in a supine position.	
	Straightens sling to remove any wrinkles. Ensures straps are not under patient.	

The above skills have been successfully demonstrated:	I agree with the Validator that I feel competent performing the above skills:
	X