



**Safe Patient Handling Equipment:
HoverJack Portable Power Supply (Battery Cart)
SKILLS CHECKLIST**

Equipment: HoverJack Battery Cart	Name (print):
Validator (print):	Date:
	Patient Care Unit/Dept.:

	<i>Performance Criteria</i>	<i>Validator Initials</i>
1.	<i>Charging the battery.</i>	
	Demonstrates pressing Battery Status switch located on the AC Power Panel on the side of the battery to check battery charge level. Demonstrates knowledge of colored indicator lights (red, amber, and green).	
	Turns off “AC Outlets” switch located on AC Power Panel above the AC outlets.	
	Plugs the AC Charger (attached to the handle of the cart behind the basket) power cord into a standard 115-volt AC outlet.	
	Plugs the AC Charger’s DC plug (cigarette lighter-type end) into the DC power socket (cigarette lighter-type receptacle) of the battery, located on the DC power panel, which is on the side of the battery opposite the AC Power Panel.	
	Turns on Battery Charger switch located on the AC Charger’s side panel.	
	Explains that while charging, the Recharge indicator on the AC Charger will illuminate amber, and will change to green when battery is fully charged.	
	Explains that the battery charger may be left connected to AC power and to the battery, as the charging voltage and current automatically reduces to a low maintenance level when the battery is fully charged.	
	Explains that the connected charger will automatically restart charging after any utility power interruption.	
2.	<i>Operating the Battery.</i>	
	Removes the AC Charger’s DC Plug from the DC power socket and disconnects the AC Charger’s power cord from the AC outlet.	
	Turns On the AC Outlets switch.	
	Plugs the AIR200G or AIR400G into one of the AC outlets on the AC Panel of the battery.	
	Turns on the Air Supply and proceeds with HoverJack operation as trained.	

The above skills have been successfully demonstrated:	I agree with the Validator that I have demonstrated the above skills: