



**Safe Patient Handling Equipment:
HoverMatt® Split-Leg
SKILLS CHECKLIST FOR LITHOTOMY POSITION**

Equipment: HoverMatt Split-Leg	Name (print):
Validator (print):	Date:
	Patient Care Unit/Dept.:

	<i>Performance Criteria</i>	<i>Validator Initials</i>
1.	Following transfer, ensures that patient is centered on OR table and deflates mattress.	
2.	Unbuckles patient safety strap.	
3.	Undoes snaps between patient's legs.	
4.	Individually moves the split legs of the mat along with the patient's legs into the desired position for the procedure.	
5.	Post procedure, reconnects the foot end attachments to the OR table, moves both split legs of mat under each of patient's legs and attaches all snaps to connect split legs together. And connects patient safety straps.	
6.	Transfers patient to stretcher.	

The above skills have been successfully demonstrated:	I agree with the Validator that I feel competent performing the above skills:
	X