

**Safe Patient Handling Equipment:
HoverMatt® PROS™ Sling**
SKILL CHECKLIST FOR BOOSTING/REPOSITIONING – AS A SLIDE SHEET

Equipment: HoverMatt PROS Sling	Name (print):
Validator (print):	Date:
	Patient Care Unit/Dept.:

	<i>Performance Criteria</i>	<i>Validator Initials</i>
	Raises the siderails.	
	To ease boosting efforts, places bed in Trendelenburg prior to boosting.	
	If product is not already in place, places PROS Sling underneath patient using a log rolling technique.	
	Ensures patient is centered on the product prior to movement.	
	Using the handles under the sling, boosts/repositions the patient using proper ergonomic positioning.	
	Puts bed back to supine.	

The above skills have been successfully demonstrated:	I agree with the Validator that I feel competent performing the above skills:
	X