

## Safe Patient Handling Equipment: Q2Roller™

### SKILLS CHECKLIST FOR LATERAL TURNING OF PATIENTS

Equipment: Q2Roller	Name (print):
Validator (print):	Date:
	Patient Care Unit/Dept.:

	<i>Performance Criteria</i>	<i>Validator Initials</i>
1.	Demonstrates placing the Q2Roller on the bed prior to receiving the patient. Tucks the securing apron flaps under each side of the mattress to hold the Q2Roller in place. Effectively layers linens and pads on top of Q2Roller per facility protocol.	
2.	Demonstrates placing the Q2Roller underneath a patient using the log-rolling technique. Tucks apron flaps and layers linens on top of Q2Roller.	
3.	Assures that the patient’s spine is aligned with the “Center Line” of the Q2Roller.	
4.	Raise the bed rails on the side of the bed toward which the patient is being turned.	
5.	Ensures that the red valve on each chamber is capped tightly.	
6.	Plugs HTAIR-1200 air supply cord into wall outlet, and verbally prepares patient for sound and sensation of Q2Roller inflation.	
7.	Locates the inflation valve that is on the opposite side of the turn direction and places the air supply hose over this valve, holding the hose in place. Pushes “ADJUSTABLE” button on the air supply keypad to initiate airflow. Demonstrates an understanding of adjustable flow by pushing the button 1-3 more times while explaining the incremental increase in airflow.	
8.	Inflates the chamber appropriately to achieve the desired patient position for pressure relief, then presses the “STANDBY” button on the air supply keypad to stop the airflow.	
9.	Demonstrates making slight adjustments in the level of inflation by depressing the center of the inflation valve, allowing air to escape the chamber.	
10.	Demonstrates the rapid deflation of the Q2Roller by removing the red cap from the deflation valve while making sure that the patient is attended and stabilized during deflation.	
11.	Demonstrates knowledge of using Q2Roller for accessing the patient’s back for wound care or hygiene: Inflates one chamber to position the patient on his or her side. Caregiver facing the patient places one hand on patient’s shoulder and one hand on patient’s hip to keep him or her in side-lying position while the other caregiver deflates the Q2Roller. The caregiver facing the patient	

	continues to hold the patient in place while the other caregiver performs the patient care.	
12.	Demonstrates knowledge of using the Q2Roller to facilitate linen change: Inflates one chamber to position the patient on his or her side. The caregiver facing the patient places one hand on patient's shoulder and one hand on patient's hip to keep him or her in side-lying position while the other caregiver deflates the Q2Roller and tucks the soiled linen underneath the patient. Applies the new sheet on top of the Q2Roller and tucks it under patient. Both caregivers gently roll the patient back to a supine position and then one caregiver inflates the opposite chamber. While the caregiver facing the patient holds the patient in place, the chamber is deflated. The soiled linens are then removed and the bed can be made with the fresh linens.	
13.	Demonstrates knowledge of 600 lb. weight limit and that the Q2Roller is a Single Patient Use device.	

The above skills have been successfully demonstrated (Validator):	<b>I agree with the Validator that I have successfully demonstrated the above skills:</b>
X	X