

HOVERMATT® PROSWedge™

Off-Loading • Slip-Resistant • Comfortable Support




Identify

Locate the patient's sacrum and assess their current position. Proper positioning begins with visualizing the sacrum and patient's position in bed.



Place

Place one wedge below the sacrum with its top edge aligned at the greater trochanter. Position the second wedge above the sacrum, leaving a hand's width of space between the two wedges. The sacrum should be fully offloaded and floating.

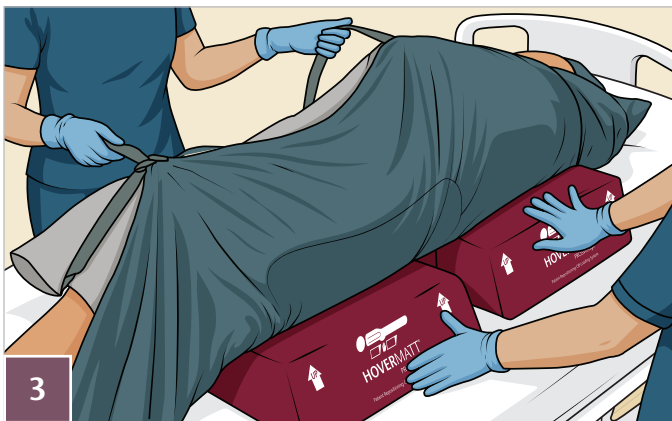

Confirm

Visually check the sacrum is fully offloaded. Reposition if needed to ensure both comfort and effective offloading.



Maintain

Reassess regularly to maintain sacral offloading and prevent skin breakdown.



To learn more about the **HOVERMATT® PROSWedge™** and our full suite of air and non-air assisted transfer devices, scan the QR code or visit hovertechinternational.com.